

## ***CLEANING INSTRUCTIONS FOR YOUR NEW DAY PRODUCTS BREAST FORMS***

**MTT, MTO, MST, MSO** - Breast forms should be cleaned after every use. Warm gentle soapy water will do the trick. All forms can be totally submerged in warm soapy water. Forms with Silicone can be towel dried and put back in their box "cradle" to keep their shape.

**MVT** - Clean your form daily in warm soapy water. Towel dry the front silicone portion. The fabric backing is all that will need a short time to dry. Just return the form to its cradle (box) to air dry. The microbeads do not absorb water. If in a hurry, you can use a blow dryer on **COOL** only to dry the fabric. Keep it 12 inches away. **Please note, the MVT form is not to be used swimming. Due to the complexity of this form the chlorine and salt water will break down the stitching.** Cleaning the form in warm soapy water will not hurt the breast form.

**MGT** - Foam forms can be washed in warm soapy water. The foam forms will retain some water, blot with a towel to help speed the drying process. Set out on a towel to dry.

**NCW2** - Beaded puffs should be washed in warm soapy water. The microbeads will not absorb water. Only the fabric cover will need to dry. If you use this form for swimming, please take it out of your swimsuit and clean it after each use.